

Drug Delivery System – DUPLEX container

## Two-Compartment Container



DUPLEX containers must be activated prior to administration. When you are ready:

				
<b>"PEEL"</b> Peel foil strip from drug chamber.	<b>"SNAP"</b> Fold container just below diluent meniscus and squeeze until seal between diluent and powder snaps open.	<b>"SHAKE"</b> Shake diluent and powder thoroughly to mix.	<b>"SNAP"</b> Squeeze folded container just below the solution meniscus to snap the second seal and release liquid into the port.	<b>"GO"</b> Remove foil tab from set port and spike bag. Hang bag on IV pole.

Excerpt from: Meropenem for Injection in B. Braun's DUPLEX delivery system. 2023.